



# HOFFMANN'S GLASWACHS

## Klebewachs gelb/rot

---

---

### Gebrauchsanweisung für den Zahnarzt / Zahntechniker

Zusammensetzung: Glaswachs enthält: Kolophonium, Bienenwachs, Carnaubawachs

#### Anwendungsgebiete:

Glaswachs gelb: Klebewachs zum Zusammensetzen von Kronen- und Brückenarbeiten.

Glaswachs rot: Klebewachs zum Ansetzen von Zähnen.

Verarbeitungsempfehlung: Das Glaswachs wird mit Hilfe des Bunsenbrenners erwärmt und auf die zu klebende Stelle geträufelt.

Packungsgröße: 14 Stangen - 70 g

### Hoffmann's Glaswachs

- nach Ablauf des Verfalldatums nicht mehr anwenden
- Farbschwankungen haben keinen Einfluß auf die Qualität

---

HOFFMANN DENTAL MANUFAKTUR GMBH, Wangenheimstraße 37/39,  
14193 Berlin, Germany, [www.hoffmann-dental.com](http://www.hoffmann-dental.com)



# HOFFMANN'S ADHESIVE WAX

## Adhesive wax yellow/red

---

---

### Instruction for the dentist/ dental technician

Composition: adhesive wax contains: colophony, beeswax, carnauba wax

#### Area of Application:

Adhesive wax yellow: for the assembly of crown and bridge work.

Adhesive wax red: for the fixation of teeth

Processing Recommendation: Heat the adhesive wax with a Bunsen burner and permit it to drip onto the assembly point.

Package Size: Package with 14 sticks, 70 g

### Hoffmann's Adhesive Wax

- Do not use after the expiration date
- minor colour variations have no effect on the quality of the product

---

HOFFMANN DENTAL MANUFAKTUR GMBH, Wangenheimstraße 37/39,  
14193 Berlin, Germany, [www.hoffmann-dental.com](http://www.hoffmann-dental.com)