HOFFMANN'S PROVISORISCHER CEMENT

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HOFFMANN'S PROVISIONAL CEMENT

Instruction for the dentist

Composition: powder contains: zinc oxide, aluminium hydroxide;
liquid contains: o-phosphoric acid

Areas of Application: Temporary fillings

Special Information: Do not apply Provisional Cement in case of:
Direct application to dentin near the pulp or to exposed pulp.

Side effects: Irritations of the pulp may occur when used as a temporary filling.

Dispensing Instructions: 1.3 g powder : 1.0 g liquid

Processing Recommendations: A clean, dry glass plate at 18 - 20°C and a non-discolouring spatula are recommended.

Mixing: Divide the measured quantity of powder into individual portions in order to achieve a homogenous consistency. Begin with the smallest quantity, quickly mix the powder into the measured liquid, a portion at a time. Discard remaining powder or impure liquids. Dentin areas near the pulp must be protected.

Mixing Time: 45 seconds

Working Time: 1 minute (at 23°C)

Net Setting Time: 1 minute 45 seconds - 3 minutes 45 seconds (at 37°C). The times indicated apply from the end of mixing. The cement is to be applied to cavity.

Notes:

- Close containers immediately after removing material.
- The liquid contains phosphoric acid and is corrosive. In the event of accidental eye contact, rinse immediately with large quantities of water.

Package Sizes: Powder: 50 g, liquid: 40 ml

Hoffmann's Provisional Cement

- Do not use after the expiration date

HOFFMANN DENTAL MANUFAKTUR GMBH, Wangenheimstraße 37/39,
14193 Berlin, Germany, www.hoffmann-dental.com