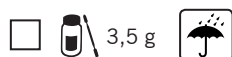
HOFFMANN'S ENDO REPAIR

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Poudre: 3,5 g, Liquide: 10 ml

HOFFMANN'S
Dental materials inspired by nature

HOFFMANN'S
ENDO REPAIR

HOFFMANN'S ENDO REPAIR

Biological bone cement on pure calcium phosphate basis

en

HOFFMANN'S ENDO REPAIR

(formerly Proxipulpine 4P)

Instruction for use for the dentist

Composition:

Powder contains: calcium phosphates, hydroxyapatite
Liquid contains: distilled water

Areas of application:

For direct and indirect pulp capping after full or partial caries removal in cases of initial pulpitis, in cases of caries profunda and iatrogenic injury of dental pulp. For the treatment of open apices (apexification) and root perforations as a result of excessive instrumentation. For the treatment of deciduous teeth after vital extraction.

Exact dispensing for optimal material features:

0.16 g powder : 0.10 g liquid

Recommendation for practicable dispensing with enclosed dosing spoon:

2 level spoons of powder : 3 drops of liquid

Shake the glass bottle to loosen the powder before removal. Fill the spoon and use a spatula to level it off. When doing so, do not compress the powder. The dropper of the liquid bottle must be free of any liquid.

Mixing:

For mixing we recommend Hoffmann's glass mixing slab (Order No. 82900) and Hoffmann's Spatula (Order No. 82908). The powder is mixed into the liquid to form a creamy consistency. Any remainders of mixed powder and liquid are to be discarded.

Working recommendations:

Take up the creamy mixture with a ball-shaped instrument - such as a ball plunger or a blunt probe - and coat the affected area thinly.

For the apexification or treatment of root perforations take up the creamy mixture with a lentulo spiral and apply to the area to be treated.

Mixing time: 10 seconds

Setting time: 0:30 – 2:30 minutes

Special Information:

- In the case of one or more symptoms, pulpitic pain, already necrotic pulp with or without clear pain to heat or non-response to cold test, the success rates of the treatment with Hoffmann's ENDO REPAIR decrease.
- In the case of bleeding of dental pulp, wait for hemostasis.
- The iatrogenic, accidental and restricted pulp opening should not exceed 1 mm².
- In combination with a filling, a lining with Hoffmann's Phosphate Cement is recommended. In the case of partial caries excavation a lining with Hoffmann's Copper Cement Cu⁺ or Hoffmann's Ready2Protect is recommended.
- After 1 or 2 weeks a dental follow-up is required.
- Do not use after the expiration date.

Safety instructions:

- Keep containers closed and always close immediately after use.
- Do not remove plastic container with silica gel from the powder bottle.
- Safety data sheets available at www.hoffmann-dental.com.

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