



**HOFFMANN'S**

# EasyFill

provisorische Verschlusspaste



made  
in  
Germany

CE 0482

40g

## **HOFFMANN´S EasyFill**

provisorische Verschlusspaste  
temporary filling material

### **Instruction for use**

#### **Composition:**

Zinc oxide, potassium sulfate, zinc sulfate

#### **Areas of Application:**

Hoffmann's EasyFill is high-adhesion filling material for temporary sealing of cavities.

#### **Dispensing Instructions:**

Hoffmann's EasyFill is a ready for use paste. Simply apply the required quantity of temporary filling paste into the cavity with a hand instrument. EasyFill will self-cure within 15 minutes. During setting time avoid any exposure to chewing pressure. Forbid any food consumption for at least one hour after treatment.

#### **Special information:**

- Tightly close the container immediately after removing the material.
- Store the product under moisture-free conditions.
- Remove the material using a probe or an excavator.
- It is vital that opened pulp is covered.
- Do not use after expiration date.
- Material safety data sheet (MSDS) available at [www.hoffmann-dental.com](http://www.hoffmann-dental.com)

**Package Size:** 1 jar à 40g

**Colour:** white

#### **HOFFMANN DENTAL MANUFAKTUR GMBH**

Wangenheimstr. 37/39

14193 Berlin, Germany

Tel: +49 30 82 00 990

Fax: +49 30 82 28 145

Email: [quality@hoffmann-dental.com](mailto:quality@hoffmann-dental.com)

[www.hoffmann-dental.com](http://www.hoffmann-dental.com)

**Order-No.:** 8100400

**Date of Issue:** 10/2013